

Contraindications

One or more of the following:

- ❌ Pregnancy
- ❌ Colostomy / Ileostomy
- ❌ Abdominal Aortic Aneurysm
- ❌ Small Bowel Obstruction
- ❌ Gastroparesis
- ❌ Anatomical Abnormalities Precluding Device Placement (e.g. kidney transplant, abdominal drain, pain stimulator)

Guidelines for Placement

Ask the patient to slightly lift his or her hips. Slide ColoWrap under the patient so the top of the primary band is **aligned with the lateral peak of the patient's iliac crest (Figure 1)**.



Figure 1

Sizing Guidelines

		Weight										
Height	in	cm	13-22 kg	23-31 kg	32-40 kg	41-49 kg	50-60 kg	61-84 kg	85-102 kg	103-122 kg	123-142 kg	>142 kg
	30-36"	76-91	PS	PS+XK	PS+XK							
	36-42"	92-106	PS	PS+XK	PL	PL	PL+XK					
	42-48"	107-121	PL	PL	PL	PL+XK	PL+XK					
	48-54"	122-137	PL	PL	PL	PL+XK	PL+XK					
	54-60"	138-152			S*	S*	S	M				
	>60"	>152			S*	S*	S	M	L	XL	XXL	XXL+XK